


Instructions For Use

1. Load & Lubricate

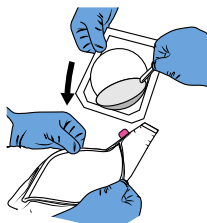

Open the funnel using the slider and remove Tyvek.

Hold the edges of the funnel open, while the surgical tech pours the implant along with 50cc of antibiotic impregnated saline (with or w/o Betadine) into the funnel.

2. Trim



Cut the tip to size and then drain the excess fluid out of the funnel.

Use the recommended markings or doctor's discretion when sizing the delivery opening.

3. Deliver



With the implant slightly crowned, insert the funnel tip past the skin edges and deliver the implant. Be mindful of the posterior implant patch orientation.

NOTE: If delivery is met with excessive resistance, cut the tip to a larger opening at surgeon's discretion.

View our
Instructional
Video





Catalog Number



Batch Code



DO NOT use the product if the package has been opened or damaged



Sterilized by Ethylene Oxide



Store at Room Temperature (18°C - 25°C)



Use By Date



Non-Phthalate



Consult the Instructions for Use



Do Not Reuse



Not made with Natural Rubber Latex



Caution



Date Manufactured



Do Not Resterilize

Rx ONLY

U.S. Federal Law restricts this device to sale by or on the order of a licensed physician



Manufacturer



Keep Dry

Description

The iNPLANT™ Funnel is a sterile, transparent, cone-shaped sleeve constructed of a flexible, Phthalate-free polymeric film and has a lubricious hydrophilic coating on the inside surface.

Intended Use

The iNPLANT™ Funnel is intended to facilitate the delivery of silicone gel and other silicone-based implants by reducing friction during insertion. It is also designed to support the implementation of a "no touch technique" during implant insertion.

Product Warranty

Proximate Concepts, LLC stands behind the iNPLANT™ Funnel . The iNPLANT™ Funnel warranty is available online at inplantfunnel.com/returns

US Patents # 9,925,028 and # 10,575,936

Other US and Worldwide Patents Pending



Proximate Concepts LLC

6 Pearl Court Suite H, Allendale NJ 07401
Phone: 201-913-8833 Fax: 201-351-5393
Email: contact@inplantfunnel.com

IFU-005 Revision.C
Effective Date: 2026-04-20