



1 PREPARE FUNNEL FOR LOADING

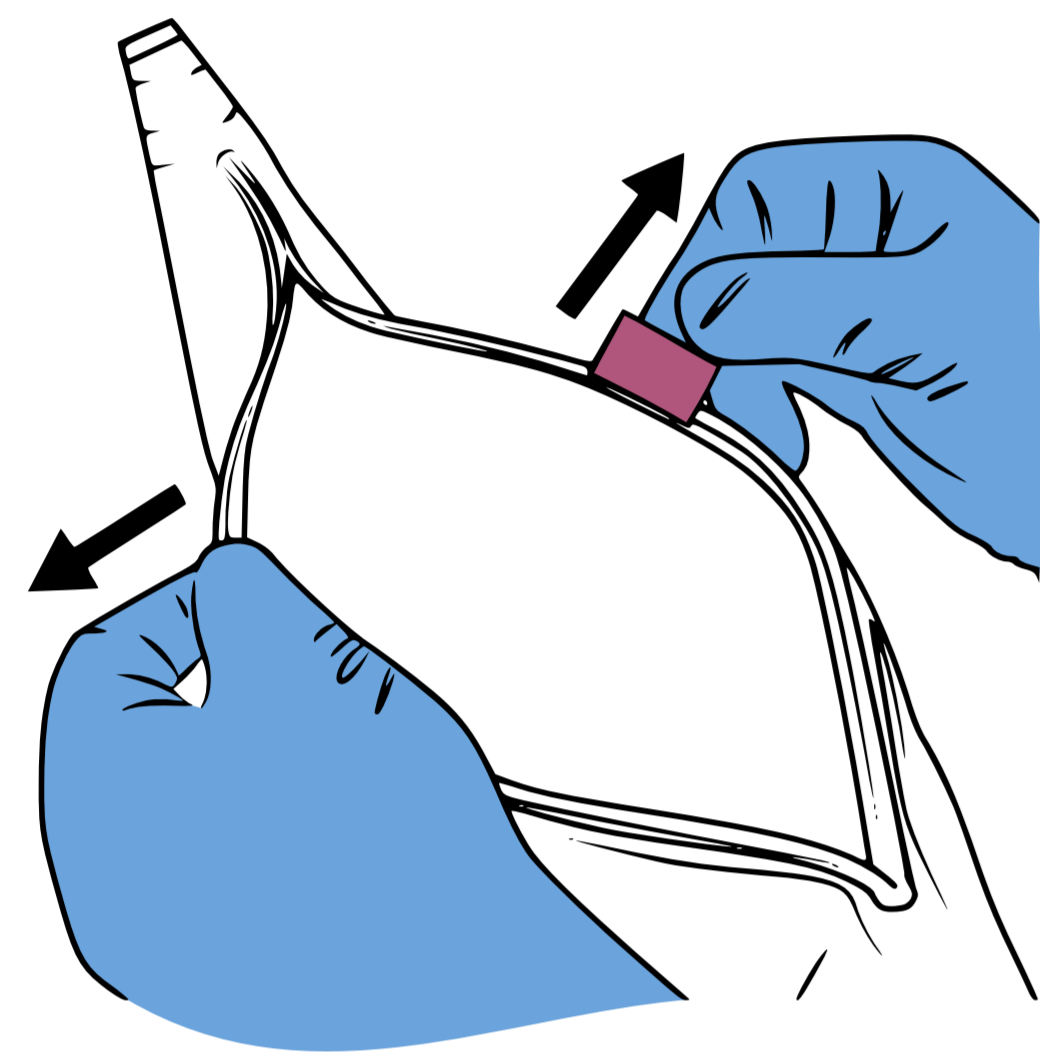

Pull pink tab to open GlideTrack for loading

Remove the Tyvek spacer and place it on the sterile field for reference

2 LUBRICATE AND LOAD

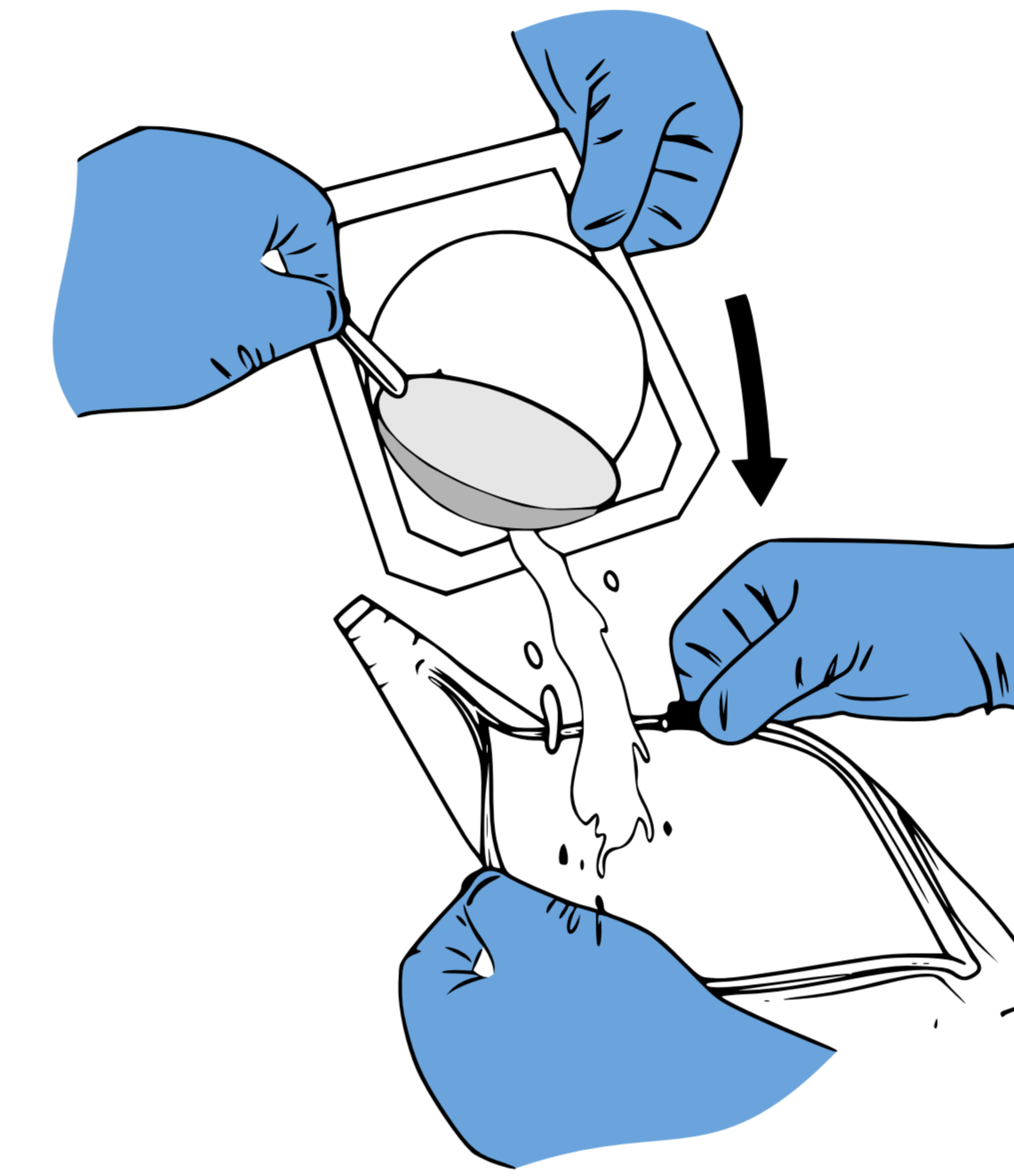


Pour 60-100 cc of solution (antibiotic solution with or without betaine) into implant packaging of both implants


Hold funnel in open position while surgical tech pours implant into the funnel

Make sure silicone tab on the implant is facing up

3 RECLOSE GLIDETRACK AND DISTRIBUTE FLUID

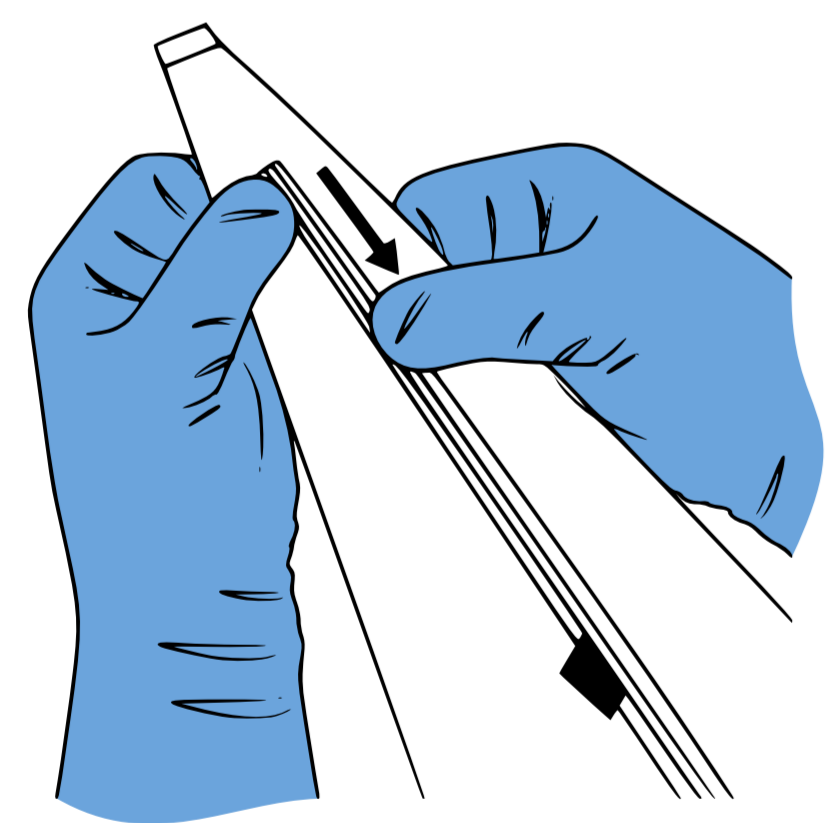


Snap and Seal the Glidetrack closed

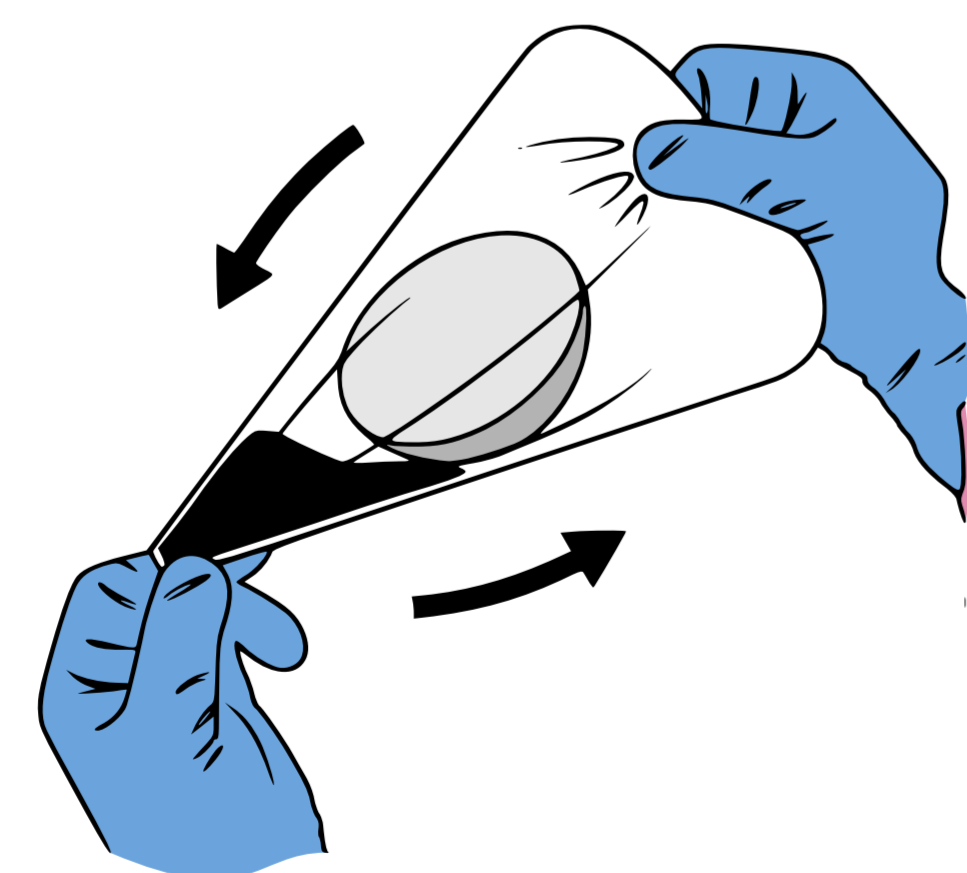


Invert the funnel once or twice to distribute the fluid

4 SIZE AND DELIVER

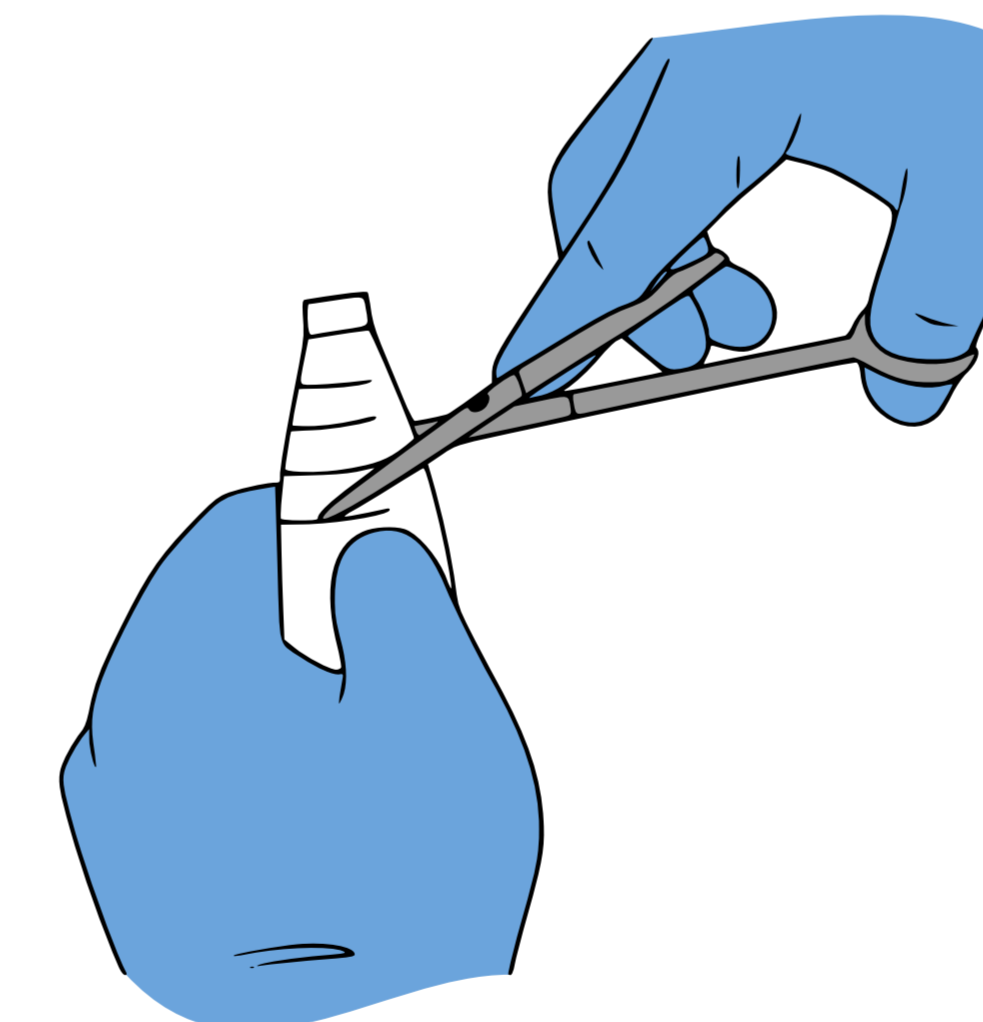

Cut tip to size and then drain the fluid out the funnel tip

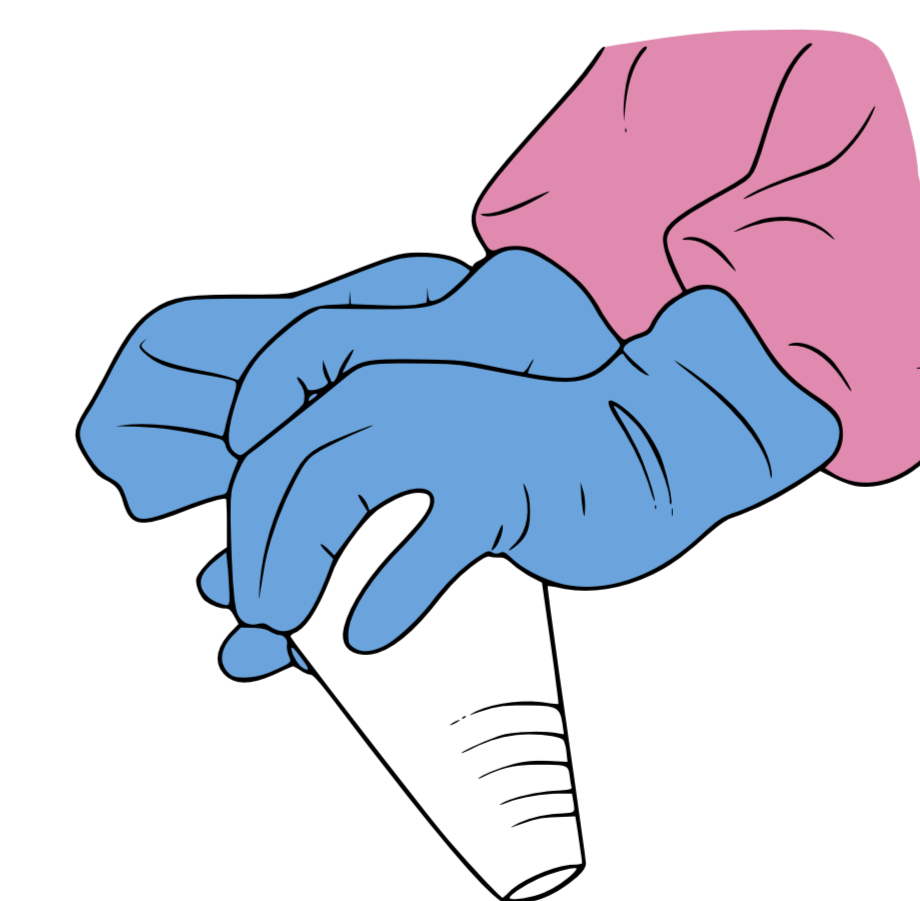

Deliver the prosthesis

NOTE - OPTIONAL STEP FOR DOUBLE LOADING

Follow previous steps to load both first and second implant in sequence before reclosing the Glidetrack. Remember that the first breast implant you place into the **iNPLANT Plus** funnel will be the second prosthesis you deliver. This is important if implants of different size are being inserted into your patient.



Proximate Concepts LLC

6 Pearl Court Suite H, Allendale NJ 07401

Phone: 201-913-8833 Fax: 201-351-5393

Email: contact@inplantfunnel.com



INTRODUCING...



Easier Opening | Stronger Materials | Double Delivery

Description

The iNPLANT™ is a sterile, transparent, cone-shaped sleeve constructed of a flexible, Pthalate-free polymeric film and has a lubricious hydrophilic coating on the inside surface.

Intended Use

The iNPLANT™ is intended to facilitate the delivery of silicone gel implants by providing a shell-tissue interface with less friction during insertion of the implant. It is also intended to facilitate implementation of a "no touch technique" in implant insertion.

Product Warranty

Proximate Concepts, LLC stands behind the iNPLANT™ Funnel. The iNPLANT™ Funnel warranty is available online at inplantfunnel.com/returns

US Patents # 9,925,028 and # 10,575,936

Other US and Worldwide Patents Pending



Catalog Number



Batch Code



Sterilized by Ethylene Oxide



Store at Room Temperature (18°C - 25°C)



Use By Date



Non-Phthalate



Do Not Reuse



Not made with Natural Rubber Latex



Date Manufactured



Do Not Resterilize



Manufacturer



Consult the Instructions for Use



Keep Dry



Caution



DO NOT use the product if the package has been opened or damaged

Rx ONLY

U.S. Federal Law restricts this device to sale by or on the order of a licensed physician